


Face Shield Assembly Instructions

1. Cut 2 inches of the Velcro and split down the middle so you will have two strips
2. Cut the bicycle tube 7 inches. Split the tube so you will have a flat rubber piece. Now cut two strips that are $\frac{3}{8}$ inches wide and 7 inches long. Attach the velcro strips to the rubber strips.
3. Cut the foam 6 inches
4. Cut out the clear shield as shown below. Round out the edges of the shield by cutting

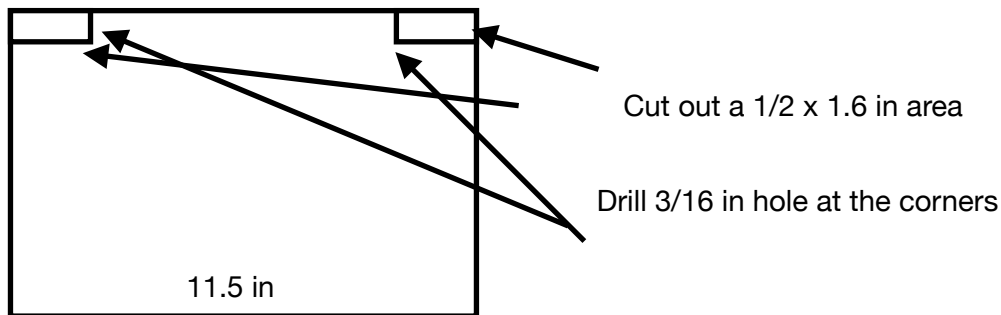

- away the plastic.
5. Center the foam and attach to the inside of the face shield head band
 6. Insert the clear plastic shield in the slot of the head band. Make the it is centered and the plastic is inserted all the way in. If you want you can temporary hold the plastic sheet in place using masking tape. Now hot glue the plastic sheet in the groove. A screwdriver can be used to separate the gap so the hot glue can flow in.
 7. Insert the rubber straps in the holes at the end of the head band. See the picture below.
 - 8.

